


# SMI-SPON

ABSORBABLE GELATIN SPONGE

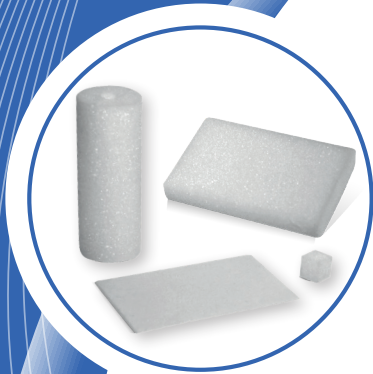


## Product highlights

- ✓ Secure and quick haemostasis, stops bleeding effectively - Significantly reduces blood loss
- ✓ Absorbs 30-40 times its own weight in blood and fluids
- ✓ Can easily be reduced in size to any required format- does not stick to gloves or instruments
- ✓ May be applied dry or be soaked with sterile physiological solution or antibiotics
- ✓ Gamma sterilized, individually blister packed and ready to use

## Benefits

- ✓ No risk of encapsulation
- ✓ Does not disintegrate after moistening
- ✓ Fasten recovery time for the patient
- ✓ Biocompatible and non pyrogenic, non-toxic, non-allergenic, non-immunogenic
- ✓ Totally absorbed in 3-4 weeks, leaves no residue and encapsulation

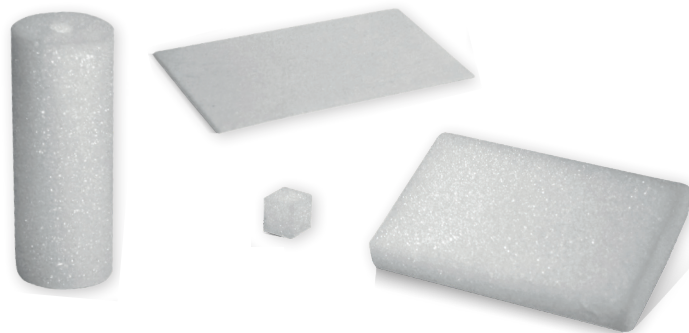


For quality  
&  
safety

**smi**  
www.sutures.be

# SMI-SPON

ABSORBABLE GELATIN SPONGE

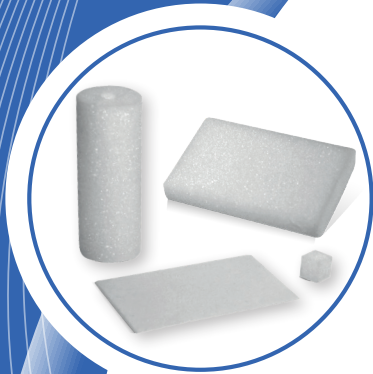

## What is SMI-SPON

SMI-SPON is a surgical haemostatic sponge, manufactured from highly purified first grade gelatin material for use in various surgical procedures.

It is a quick and effective haemostatic which stops bleeding fast. It reduces intraoperative bleeding and surgery time.

## Why using SMI-SPON

SMI-SPON can be used in various surgeries for haemostasis. When applied in vivo and used in appropriate amounts, it is completely absorbed within 3-4 weeks. SMI-SPON is biocompatible, non-toxic and non-pyrogenic.

SMI-SPON is gamma sterilized and individually packed in ready to use blister packaging.

## Where to use

- Dental surgery
- Plastic surgery
- Vascular surgery
- Orthopaedic surgery
- ENT surgery
- Neuro surgery
- Abdominal surgery
- Anorectal surgery
- Genito Urinary surgery
- Gynaecological surgery

## How to use SMI-SPON

- Cut a piece to the desired size
- Apply SMI-SPON either in a dry state or saturated with physiological salt solution
- If used dry, a piece of SMI-SPON is applied to the bleeding place and held in place with moderate pressure until hemostasis results
- If used wet, first immerse in the solution and then withdraw, squeeze between gloved fingers to expel air bubbles, and the replace in saline until needed

Product	Unit	Order no.
<b>Regular</b>	70 x 50 x 10 mm	10 Pc. <b>ZHG705010</b>
<b>Standard</b>	80 x 50 x 10 mm	10 Pc. <b>ZHG805010</b>
<b>Dental</b>	10 x 10 x 10 mm	32 Pc. <b>ZHG101010</b>
<b>Tampon</b>	80 x 30 mm DIA	6 Pc. <b>ZHG8030</b>
<b>Special</b>	80 x 50 x 1 mm	20 Pc. <b>ZHG805001</b>
<b>Film</b>	200 x 70 x 0.5 mm	10 Pc. <b>ZHG20070005</b>



*For quality  
&  
safety*

**smi**  
www.sutures.be